



Lab. Test BT_{DEA 1}

Canine result form for blood typing test

Date :

Patient name :

Identification code :

Typing performed by :

Lab Test lot number :

Blood type : **DEA 1 +** **DEA 1 -**


To interpret your result, stick the membrane below,
Respecting the orientation of the membrane




READ IMMEDIATELY AFTER TESTING

IF POSITIVE, THE DEA1 LINE WILL ALWAYS SHOW DIFFERENT INTENSITY:
WEAK, NORMAL, STRONG

INTERPRETATION OF RESULTS

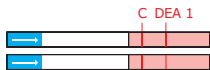

C = CONTROL LINE

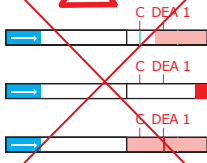

Lab.Test BT_{DEA 1}



Canine procedure for blood typing test

Provided by Alvedia :



Additional material needed :



1 buffer



1 membrane




1 result form



1 clean test tube

OR

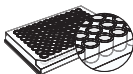

1 microplate



1 pipette

1



3


2




10 µL

*Whole blood or packed red blood cells
ACD/CPD/EDTA
Do not use Heparin

3

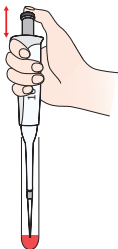

4

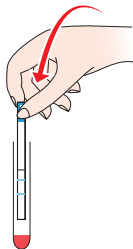

5



2 to 5 min

Wait until complete migration

6

