



Lab. Test BT_{A+B}

Feline result form for blood typing test

Date :

Patient name :

Identification code :

Typing performed by :

Lab Test lot number :

Blood type : **A** **B** **AB**

To interpret your result, stick the membrane below,
respecting the orientation of the membrane :




READ IMMEDIATELY AFTER TESTING

C = CONTROL LINE


INTERPRETATION OF RESULTS

A

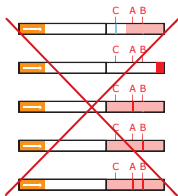

Contact us for the
«Troubleshooting
procedure»

B

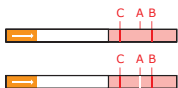

AB



Lab Test BT_{A+B}

5 min

Feline procedure for blood typing test

Provided by Alveda : 

Additional material needed :



1 buffer



1 membrane




1 result form



1 clean test tube

OR

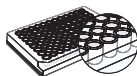

1 microplate



1 pipette


1



3


2

10 µL

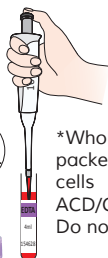


*Whole blood or packed red blood cells
ACD/CPD/EDTA
Do not use Heparin

3

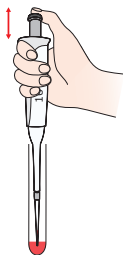

4

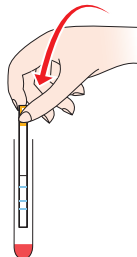

5



2 to 5 min

Wait until complete migration

6

