



# Lab Test BT<sub>DEA 1</sub>

## Canine result form for blood typing test

FIRST-TIME DONOR : Following human good practice, we recommend to perform blood typing twice using 2 kits / 2 blood samples / 2 operators.

Date : .....

Patient name : .....

Identification code : .....

Typing performed by : .....

Lab Test lot number : .....

Blood type :  **DEA 1 +**       **DEA 1 -**


To interpret your result, stick the membrane below,  
**Respecting the orientation of the membrane**



READ IMMEDIATELY AFTER TESTING

IF POSITIVE, THE DEA1 LINE WILL ALWAYS SHOW DIFFERENT INTENSITY: WEAK, NORMAL, STRONG


## INTERPRETATION OF RESULTS

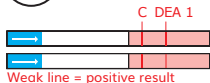

Weak line = positive result



White line = negative result

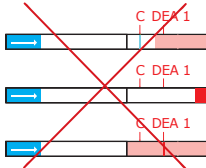

C = CONTROL LINE

# Lab Test BT<sub>DEA 1</sub>

Canine procedure for blood typing test



Provided by Alveda :



Additional material needed :



1 buffer



1 membrane



1 clean test tube

OR



1 microplate



1 pipette

1



3


2




10 µL

\*Whole blood or packed red blood cells (if anemic)  
ACD/CPD/EDTA  
Do not use Heparin

3

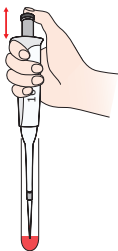

4

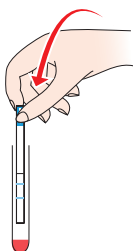

5



2 to 5 min

Wait until complete migration

6



To interpret the result, stick the membrane.

**FIRST-TIME DONOR :**

Following human good practice, we recommend to perform blood typing twice using 2 kits / 2 blood samples / 2 operators.

Scientific Service Laboratory  
[www.alveda.com](http://www.alveda.com)  
[contact@alveda.com](mailto:contact@alveda.com)