



QuickTest BT Ca

Equine result form for blood typing test

FIRST-TIME DONOR: Following human good practice, we recommend to perform blood typing twice using 2 kits / 2 blood samples / 2 operators.

Patient name :

Identification code :

Owner :

Blood type : CA+ CA-


Blood typing performed by :

Veterinarian

Date :

QT lot number :

BLOOD TYPE COMPATIBILITY

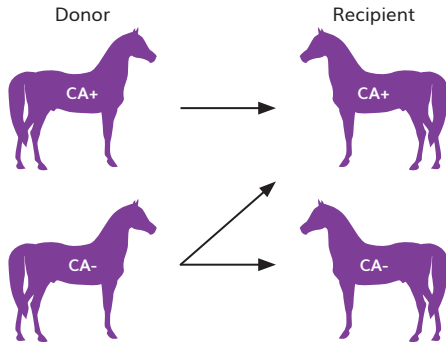

Compatible transfusions are indicated by arrows.

Before any transfusion

- Always verify CA blood compatibility between donor and recipient.
- For all other blood types : perform a crossmatch test (XM) between donor and recipient.

I agree to be a blood donor

QuickTest BT Ca



Canine procedure for blood typing test

FIRST-TIME DONOR: Following human good practice, we recommend to perform blood typing twice using 2 kits / 2 blood samples / 2 operators.




-
-
-
-
-
-
-
-

Blood tube in ACID/CPD/EDTA Do not use Heparin.
Use 10 µL of whole blood or packed red blood cells (if anemic).

Wait until complete migration
2 to 5 min.

READ IMMEDIATELY AFTER TESTING

C = CONTROL LINE

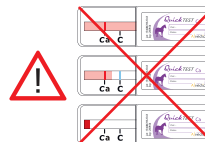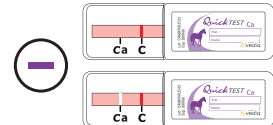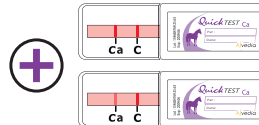

Please contact us for the «Troubleshooting procedure»

Alvedia
Alice Veterinary Diagnostic

www.alvedia.com
contact@alvedia.com