



Lab Test BT_{A+B}

Feline result form for blood typing test

Date :

Patient name :

Identification code :

Typing performed by :

Lab Test lot number :

Blood type : ☐ **A** ☐ **B** ☐ **AB**


To interpret your result, stick the membrane according to the template below.




C = CONTROL LINE

INTERPRETATION OF RESULTS


A

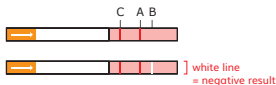

B

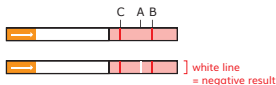

AB

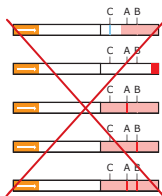

Lab.Test BT_{A+B}

5 min

Feline procedure for blood typing test

Provided by Alvedia :

Additional material needed :



1 buffer



1 membrane




1 result form



1 clean test tube

OR

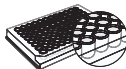

1 microplate




1 pipette

1



3


2

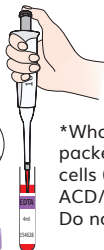


10 μ L

*Whole blood or packed red blood cells (if anemic)
ACD/CPD/EDTA
Do not use Heparin

3

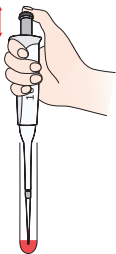

4

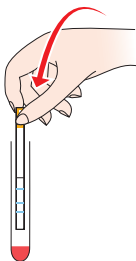

5



2 to 5 min

Wait until complete migration

6



To interpret the result, stick the membrane.

FIRST-TIME DONOR :

Following human good practice, we recommend to perform blood typing twice using 2 kits / 2 blood samples / 2 operators.

Scientific Service Laboratory

www.alvedia.com
contact@alvedia.com