



# QuickTest BT A+B

Feline result form for blood typing test

Patient name : .....

Identification code : .....

Owner : .....

Blood type :  A  B  AB


Blood typing performed by : .....

Date : .....

QT lot number : .....

Veterinarian

## BLOOD TYPE COMPATIBILITY

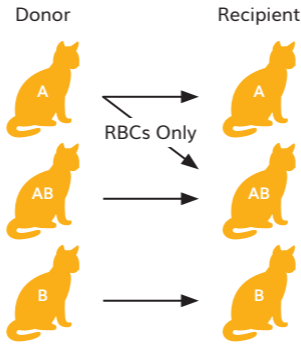

Compatible transfusions are indicated by arrows.

### Before any transfusion

- Always verify AB blood compatibility between donor and recipient.
- For all other blood types : perform a crossmatch test (XM) between donor and recipient.

I agree to be a blood donor





# QuickTest BT A+B




## Feline procedure for blood typing test



Procedure  
& Test Reader

**FIRST-TIME DONOR:** Following human good practice, we recommend to perform blood typing twice using 2 kits / 2 blood samples / 2 operators.

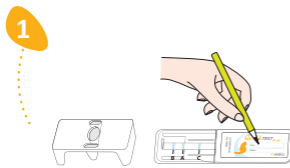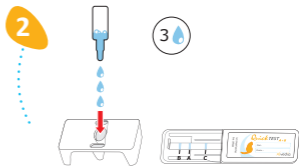


- 
- 
- Blood tube in ACD/CPD/EDTA  
Do not use Heparin.**

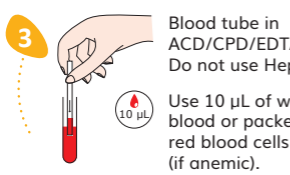**Use 10  $\mu$ L of whole blood or packed red blood cells (if anemic).**

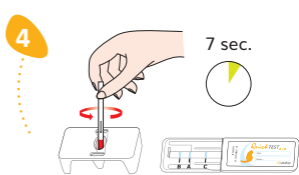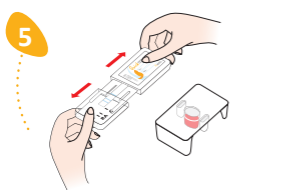
- 7 sec.**


- 
- Wait until complete migration**

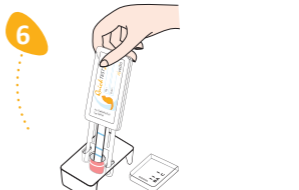**2 to 5 min.**


- 
- 

### READ IMMEDIATELY AFTER TESTING

C = CONTROL LINE

**A**




WHITE LINE = NEGATIVE RESULT

**B**



WHITE LINE = NEGATIVE RESULT

**AB**

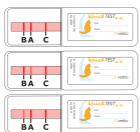

WEAK LINE = POSITIVE RESULT



Please contact us for the «Troubleshooting procedure»

**Alvedia**  
Alice Veterinary Diagnostic

[www.alvedia.com](http://www.alvedia.com)  
[contact@alvedia.com](mailto:contact@alvedia.com)