



Lab Test BT_{DEA 1}

Canine result form for blood typing test

Date :

Patient Name :

Identification code :

Typing performed by :

Lab Test lot number :




Blood Type : **DEA 1 +** **DEA 1 -**

To interpret your result, stick the membrane according to the template below.



C = CONTROL LINE

INTERPRETATION OF RESULTS

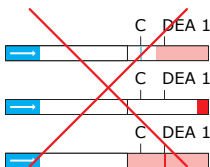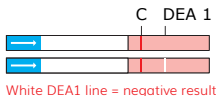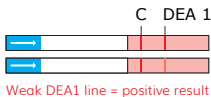

Contact us for the «Troubleshooting procedure»

Alvedia
Alice Veterinary Diagnostic

Procedure Alvedia - maj 14/12/2022 - SmallStories®



Procedure
& Test Reader

Lab.Test BT_{DEA 1}

Canine procedure for blood typing test



Provided by Alvedia :

Additional material needed :



1 buffer



1 membrane



1 result form



1 clean test tube

OR



1 microplate




1 pipette

1



3


2

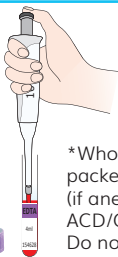


10 µl

*Whole blood or packed red blood cells (if anemic)
ACD/CPD/EDTA
Do not use Heparin

3

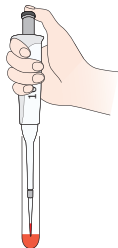

4

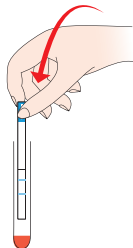


5



2 to 5 min

Wait until complete migration

6

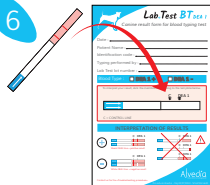

To interpret the result, stick the membrane.

FIRST-TIME DONOR:
Following human good practice, we recommend to perform blood typing twice using 2 kits / 2 blood samples / 2 operators.

Scientific Service Laboratory
www.alvedia.com
contact@alvedia.com