

Lab Test BT_{DEA1}

Canine procedure for blood typing test



ALVEDIA
alice veterinary diagnostics

Provided by Alvedia :



20 membranes



1 buffer



20 result forms

Additional material needed :



1 test tube



1 pipette

1



SCAN the QR code

1. Test procedure
2. Test reader
3. E-recording
4. Blood typing card
5. Scientific assistance
6. Troubleshooting

2



Only remove the blue cap




3

3




*EDTA whole blood
or packed red blood
cells (if anemic)
Or ACD/CPDA
Do not use Heparin

4

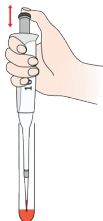

7 sec.

5

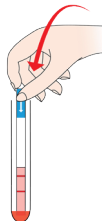


2 to 5 min.

6

RESULTS

Scan your test

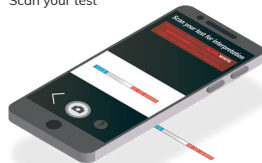

OR

USE THE RESULT FORM
ON THE BACK



LabTest BT_{DEA1}

Canine result form for blood typing test

Date :

Patient Name :


Identification code :

Typing performed by :

Lab Test lot number :

Blood Type : ☐ **DEA 1 +** ☐ **DEA 1 -**

To interpret your result, stick the membrane according to the template below.

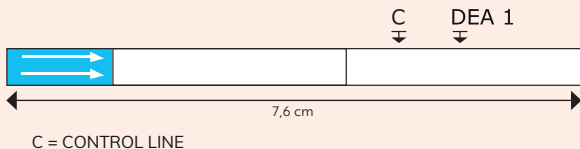


INTERPRETATION OF RESULTS



Weak DEA1 line = positive result



White DEA1 line = negative result

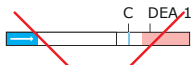

Contact us for the « Troubleshooting procedure »

