

Lab Test BT A+B

Feline procedure for blood typing test



ALVEDIA
alice veterinary diagnostics

Provided by Alvedia :



20 membranes



1 buffer



20 result forms

Additional material needed :



1 test tube



1 pipette

1



SCAN the QR code

1. Test procedure
2. Test reader
3. E-recording
4. Blood typing card
5. Scientific assistance
6. Troubleshooting

2




3




*EDTA whole blood
or packed red blood
cells (if anemic)
Or ACD/CPDA
Do not use Heparin.

4

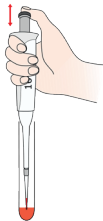

7 sec.

5

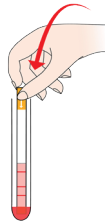


2 to 5 min.

6

RESULTS

Scan your test

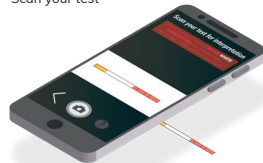

OR

USE THE RESULT FORM
ON THE BACK



Lab Test BT A+B

Feline result form for blood typing test

Date :

Patient Name :


Identification code :

Typing performed by :

Lab Test lot number :



Blood Type : ☐ **A** ☐ **B** ☐ **AB**

To interpret your result, stick the membrane according to the template below.

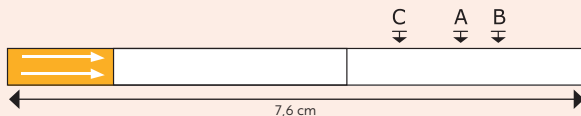

C = CONTROL LINE

INTERPRETATION OF RESULTS

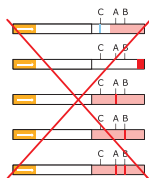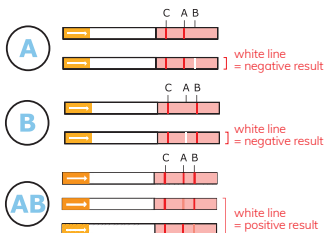

Contact us for the
« Troubleshooting procedure »