

QuickTest BT_{DEA1}

Canine procedure for blood typing test




1 SCAN the QR code




1. Test procedure
2. Test reader
3. E-recording
4. Blood typing card
5. Scientific assistance
6. Troubleshooting

2

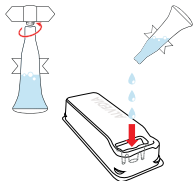


3

3

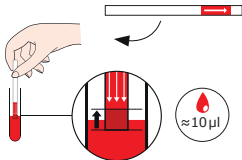


*EDTA whole blood or packed red blood cells (if anemic). Or ACD/CPDA. Do not use Heparin.

4

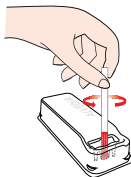

7 sec.


5

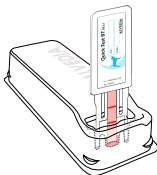

2 to 5 min.

6


Scan your test for app interpretation.

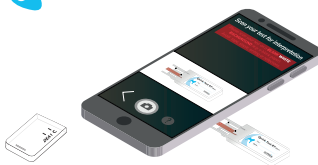

OR

7

Operator interpretation

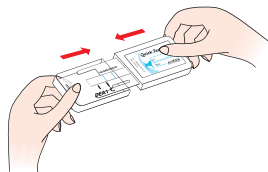


RESULTS

C = CONTROL LINE



WEAK DEA1 LINE = POSITIVE RESULT

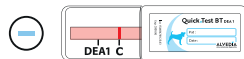

In some cases : white line in front of
DEA1 = NEGATIVE