

QuickTest BT_{A+B}

Feline procedure for blood typing test




1 SCAN the QR code




1. Test procedure
2. Test reader
3. E-recording
4. Blood typing card
5. Scientific assistance
6. Troubleshooting

2

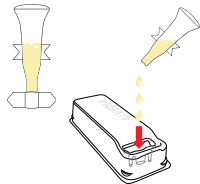

3


3

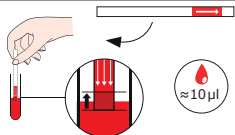

~ 10 µl


*EDTA whole blood or packed red blood cells (if anemic).
Or ACD/CPDA. Do not use Heparin.

4

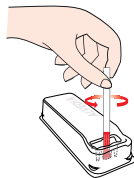

7 sec.


5

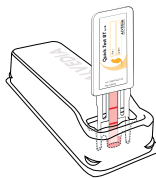

2 to 5 min.

6


Scan your test for
app interpretation

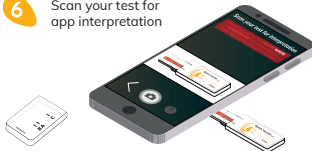

OR

7

Operator interpretation

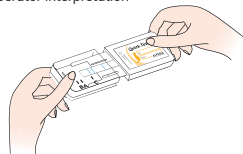

RESULTS

C = CONTROL LINE

A



B



AB



WEAK A OR B LINE = POSITIVE RESULT

In some cases : white line in front of A or
B = NEGATIVE

