

QUICK PROCEDURE GEL TEST MAJOR XM

Same procedure for Canine, Feline & Equine



SCAN ME :
Movie procedure
Test result
Record result
Troubleshooting



5 min
HANDLING



10 min
INCUBATION




10 min
CENTRIFUGATION

What we provide




1 buffer
solution

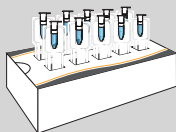

1 box of 10 XM
Gel Tests

What you need

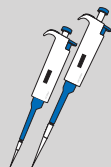

2 micropipettes
(10-100 μ l + 100-1 000 μ l)



2 test tubes (min. 1,5 ml)
for 1 XM Test



Specific centrifuge*

Blood samples

DONOR
pRBCs or
segment**




RECIPIENT
Plasma



EDTA, CPDA or ACD ONLY
Do not use heparin


** SEGMENT > 7 DAYS : WASH 1 TIME

Centrifuge and :

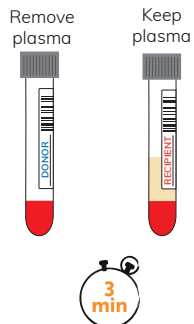

Drucker : Program
"Blood separation"
Hettich : 3500 RPM

Centrifuge :

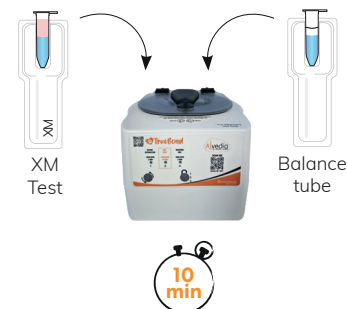

Drucker : Program "Gel Test"
Hettich : 1500 RPM