



SCAN ME :
Movie procedure
Test result
Record result
Troubleshooting



5 min
HANDLING



10 min
INCUBATION




10 min
CENTRIFUGATION

What we provide




1 buffer
solution

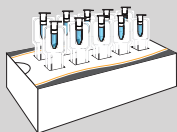

1 box of 10 XM
Gel Tests

What you need

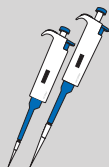

2 micropipettes
(10-100µl + 100-1000µl)



2 test tubes (min. 1,5ml)
for 1 XM Test



Specific
centrifuge*

Blood samples

DONOR
Plasma
or plasma
segment




RECIPIENT
pRBCs




EDTA, CPDA or ACD ONLY
Do not use heparin

Centrifuge and :

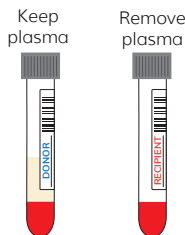

Drucker : Program
"Blood separation"
Hettich : 3500 RPM

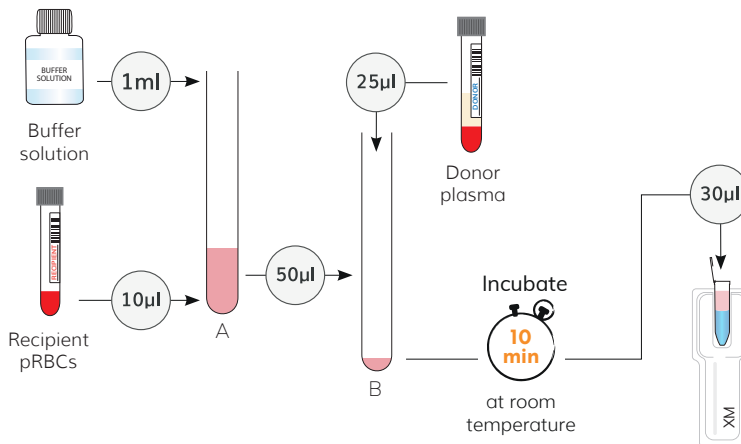

Centrifuge :



Drucker : Program "Gel Test"
Hettich : 1500 RPM